


DuraLite™ Foot

Patients who need a very lightweight foot with high-energy return and prosthetists who want a foot that is easily field-height adjustable now have the ideal choice with the DuraLite™ Foot. The foot incorporates a urethane toe pad which enhances medial and lateral motion. The DuraLite's split heel conforms to the floor and provides stability on uneven terrain. The choice of a pyramid adaptor or pyramid receiver is offered for a proximal adaptor. A unique feature of this foot is a Stair Guard. Mounted immediately beneath the proximal adaptor, the Stair Guard helps to reduce the risk of the shin adaptor catching on a step when descending stairs.

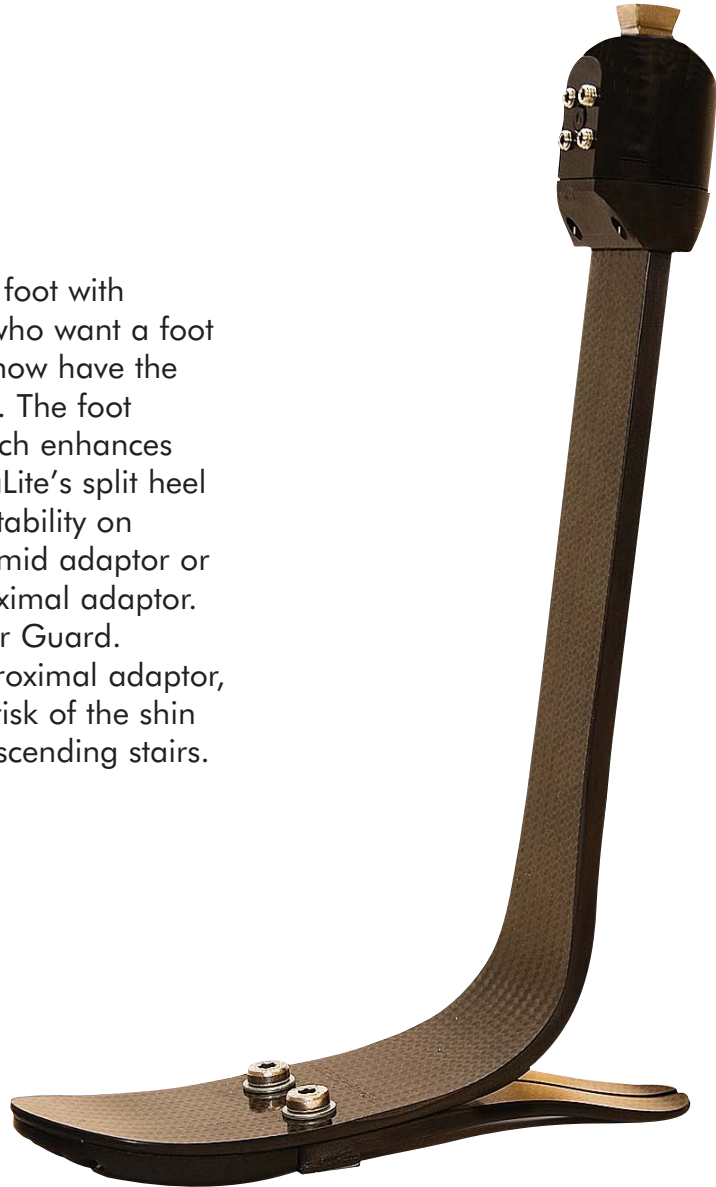

e-mail sales@ortho-europe.co.uk

www.ortho-europe.com

Tel: 01420 83294 Fax: 01420 80068

DuraLite™ Foot

Patients who need a very lightweight foot with high-energy return and prosthetists who want a foot that is easily field-height adjustable now have the ideal choice with the DuraLite™ Foot. The foot incorporates a urethane toe pad which enhances medial and lateral motion. The DuraLite's split heel conforms to the floor and provides stability on uneven terrain. The choice of a pyramid adaptor or pyramid receiver is offered for a proximal adaptor. A unique feature of this foot is a Stair Guard. Mounted immediately beneath the proximal adaptor, the Stair Guard helps to reduce the risk of the shin adaptor catching on a step when descending stairs.

The DuraLite is a lightweight responsive foot ideal for normal daily activities



- Moderate to High Activity (U.S. Levels 3)
- Adjust Shank to Suit Patient Needs
- Exceptionally Lightweight
- 300 lb Weight Limit

Specifications

INCLUDES	High-Activity Foot Shell, Spectra Sock, Pyramid Receiver or Rotating Pyramid, 1 of 6 Toe Spring Options, 3 Heel Wedges
WEIGHTS	Weight 356 g*
adaptor WEIGHTS	Pyramid 130 g; Receiver 147g
PATIENT WEIGHT LIMIT	Foot Size 23-24 cm: 104 kg** Foot Size 25-26 cm: 122 kg** Foot Size: 27-31 cm: 135 kg**
HEEL HEIGHT	10 mm or 18 mm
CLEARANCE	13" (330 mm) maximum 5.9" (150 mm) for 10mm heel 6.3" (160 mm) for 18mm heel
AVAILABLE SIZES	23 to 31 cm
TOE RESISTANCE	Six - See Composite Toe Spring Chart
HEEL SPRING OPTIONS	Three Wedges (included with each foot)
COSMESIS	Lifelike in Buff, Tan or Medium Brown



e-mail sales@ortho-europe.co.uk Cosmesis

www.ortho-europe.com
Tel: 01420 83294 Fax: 01420 80068

ORDERING GUIDELINES

To order a DuraLite Foot, please select the size, side, color, toe spring stiffness, and heel height.

	DLF-23LB-4L-1	DuraLite Foot SAMPLE Part Number
Name Code		DLF = DuraLite Foot
Size of Foot	DLF	23 to 31 cm
Side of Foot	23	L = Left
	L	R = Right
Foot Shell Color	B	B = Buff
		M = Medium Brown (special order)
		T = Tan (special order)
Toe Spring Stiffness	4	1 to 6 (refer to chart below)
Heel Height	L	L = Low 10 mm
		M = Medium 18 mm
Proximal adaptor	1	1 = Pyramid adaptor
		2 = Pyramid Receiver

GENERAL GUIDELINES

To Select Proper Composite Toe Spring Stiffness:

1. Calculate adjusted body weight. (Adjusted body weight = patient body weight + loads **normally or routinely** carried)
2. Using adjusted body weight and patient activity level, select the toe spring stiffness from the chart below.

Composite Toe Spring Stiffness Chart

		Foot Size		
		23 -24 cm	25-26 cm	27-31 cm
100-120 lbs (45-54 kg)	Medium Activity	Category 1	Category 1	Category 1
	High Activity	Category 2	Category 2	Category 2
121-150 lbs (55-68 kg)	Low Activity	Category 1	Category 1	Category 1
	Medium Activity	Category 2	Category 2	Category 2
	High Activity	Category 3	Category 3	Category 3
151-190 lbs (69-86 kg)	Low Activity	Category 2	Category 2	Category 2
	Medium Activity	Category 3	Category 3	Category 3
	High Activity	Category 4	Category 4	Category 4
191-230 lbs (87-104 kg)	Low Activity	Category 3	Category 3	Category 3
	Medium Activity	Category 4	Category 4	Category 4
	High Activity	Category 5	Category 5	Category 5
231-270 lbs (105-122 kg)	Low Activity		Category 4	Category 4
	Medium Activity		Category 5	Category 5
	High Activity			Category 6
271-300 lbs (123-135 kg)	Low Activity			Category 5
	Medium Activity			Category 6