

TITANIUM TUBE CLAMP ADAPTOR

6PC558

Instructions for Use



ASSEMBLY AND FITTING

1. Degrease the inside of the tube clamp fitting with a suitable cleaning agent such as acetone.
2. Do not handle the cleaned areas.
3. Completely remove the clamping screw.
4. Remove any previously applied thread locking agent from both the screw and the thread of the tube clamp.
5. Clean both with a suitable cleaning agent.
6. Assemble the mating tube into the tube clamp fitting.
7. Apply thread locking 243 to the screw thread in accordance with the manufacturer's instructions.
8. Re-assemble the clamping screw into the tube clamp fitting and torque tighten to torque shown.
9. Do not exceed the clamping torque.

Alignment can be retained by adjusting 2 adjacent screws.

TECHNICAL DATA

Product Weight	Patient Weight Limit	Activity Level
79g	125kg	High



WARRANTY

Components are covered by 12 months limited warranty.
 The warranty requires fitting and use as set out in these instructions.
 The warranty is void if fitting and routine care instructions are not followed.

(6PC558 Issue 2)

