

ALUMINIUM 400MM TUBE AND CLAMP ASSEMBLY

6PC570

Instructions for Use



DESCRIPTION

Adaptor and aluminium tube.

ASSEMBLY AND FITTING

To cut aluminium tube

1. Important! Do not scribe lines or damage aluminium tube. Use masking tape and pen.
2. Ensure that tube end is cut squarely and deburred.

To assemble to any tube clamp

1. Degrease the end of the aluminium tube with suitable cleaning agent such as acetone.
2. Do not handle the cleaned area.
3. Follow clamp screw assembly as set out for relevant part.

Corrosion protection

Where the aluminium tube has been shortened the cut ends must be treated with a suitable anti-corrosion compound e.g. clear acrylic lacquer.

PLANNED PREVENTATIVE MAINTENANCE

Check for damage to tube surface e.g. dents, scratches, corrosion. Replace if damage or corrosion is present.

Alignment can be retained by adjusting 2 adjacent screws


TECHNICAL DATA

Product Weight	Patient Weight Limit	Activity Level
306g	100kg	High

Precautions

Do not clamp the tube in a vice when cutting. A cutting jig (U10309) must be used at all times.



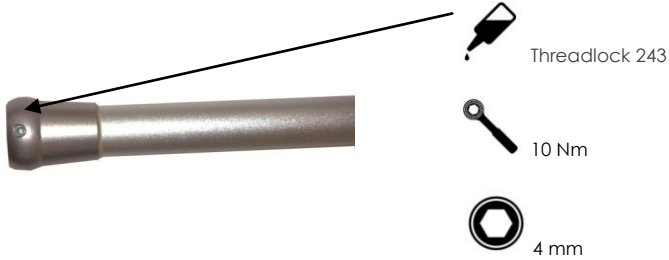

WARRANTY

Components are covered by 12 months limited warranty.
The warranty requires fitting and use as set out in these instructions.
The warranty is void if fitting and routine care instructions are not followed.

(6PC570 Issue 2)





www.ortho-europe.com • info@ortho-europe.com • +44 (0)1235 552 895
Ability House, Nuffield Way, Abingdon, OX14 1RL