

TITANIUM 100MM TUBE WITH PYRAMID RECEIVER

6PC583

Instructions for Use



ASSEMBLY AND FITTING


To Assemble to any Tube Clamp

1. Degrease the end of the tube with suitable cleaning agent such as acetone.
2. Do not handle the cleaned area.
3. Follow clamp screw assembly as set out for relevant part.

Alignment can be retained by adjusting 2 adjacent screws

TECHNICAL DATA

Product Weight	Patient Weight Limit	Activity Level
128g	125kg	High

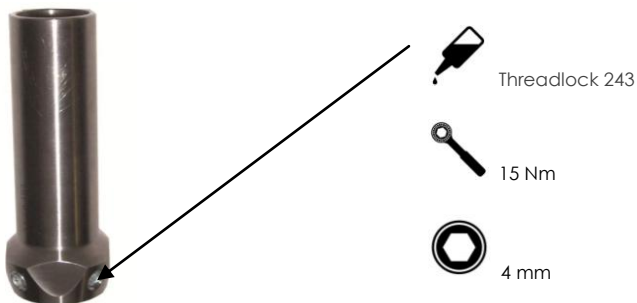

WARRANTY

Components are covered by 12 months limited warranty.
The warranty requires fitting and use as set out in these instructions.
The warranty is void if fitting and routine care instructions are not followed.

(6PC583 Issue 2)

