

TITANIUM DOUBLE ADAPTOR

6PC630
(32mm)

Instructions for Use




ASSEMBLY AND FITTING

Alignment can be retained by adjusting 2 adjacent screws.
For spare parts use the part numbers below:


Part Number	Screw Length
D0165 M8x8	8mm
D0176 M8x10	10mm
D0167 M8x12	12mm
D0177 M8x16	16mm

TECHNICAL DATA


Product Weight	Patient Weight Limit	Activity Level
89g	150kg	High

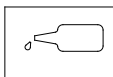

Threadlock 243

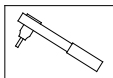

Torque – 15Nm

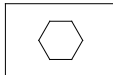

Hex Key – 4mm

WARRANTY

Components are covered by 12 months limited warranty.

The warranty requires fitting and use as set out in these instructions.

The warranty is void if fitting and routine care instructions are not followed.

(6PC586 Issue 2)

