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## INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

## FUNCTION

EPIDYN-STABIL® relieves and stabilizes the elbow. The elastic, compressive fabric and the lateral silicon pads improve proprioception and metabolism. Damaged tissue is decongested and specifically stimulated for regeneration. The elbow is stabilized using the lateral spiral springs. The collateral ligament-stabilizing springs reduce irritation to the tendon origin and additionally counteract pain.

## INDICATIONS

- Epicondylitis humeri radialis
- Epicondylitis humeri ulnaris
- Collateral ligament instabilities

## CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

– Arterial vasculopathy.

– Recent traumatic instabilities.

So far no consequential incompatibilities or allergic reactions have been reported.

## SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

## FITTING

- EPIDYN-STABIL® can be worn on the left or right.
- Pull the bandage over your elbow. The orange-colored stripe must be on your upper arm; the seam must be on the outside of the elbow.
- Crook of the arm and elbow rest centrally in the bandage, in the area of the knitted-in marking.
- The spiral springs run laterally over the inner and outer joint protrusion.

– 1

– The Velcro closure on the upper bandage edge can only

be closed if the underlay beneath is cut to fit. This has the functionality of preventing direct contact to the skin by the Velcro closure in the case of a larger upper arm circumference. When the bandage is tightened, take the measurement for the Velcro closure/underlay and allow the underlay to overlap the overlay surface slightly. Mark this point.

– 2

- Remove the bandage and cut the underlay at the marked point.

– The bandage can now be reapplied and the Velcro closure closed on the upper bandage edge.

## IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

Please close the Velcro fastener properly if the device has to be returned to the manufacturer.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

## QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

## CARE

Wash EPIDYN-STABIL® in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. Please close the Velcro fastener when cleaning, etc.

**TIP:** If the tension in the anatomically-contoured knit slackens over time due to wear, it can be restored by washing. We therefore recommend washing the bandage frequently.

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Alla produkter från SPORLASTIC GmbH genomgår produktkontroll inom vårt kvalitetshanteringssystem.