EPIDYN®
SPORLASTIC

EPIDYN AKTIV®
SPORLASTIC

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67% Polyamid, 33% Lycra

INSTRUCTIONS FOR USE GB

Dear Customer, we would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest stockist or contact us directly.

FUNCTION

EPIDYN® and EPIDYN AKTIV® relieve and compress the elbow joint. The integrated friction pressure pad with its knopped surface towards the skin has a positive effect on painful insertion tendopathy, capsular irritation and epicondylopathy, which are common in arthrosis. The improved circulation speeds up the absorption of effusions. The circular EPIDYN AKTIV® band provides better local strain relief for inflamed tendons of the epicondyle and accelerates the healing process.

INDICATIONS

- Epicondylitis
- Tendomyopathy
- Arthrosis of the elbow joint
- Ligament insufficiency
- Concomitant epicondylopathy with arthrosis
- Elbow distortion
- Irritations following operations, fractures, rheumatic elbow effusions

KONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING

- EPIDYN® and EPIDYN AKTIV® can be worn on either the left or right elbow.
- For EPIDYN AKTIV®, open the circular band.
- Pull the bandage over your elbow. The orange stripe should be located against your upper arm, the seam on the outside of the elbow.
- Position the ring-shaped depressions of the silicon insert so that they rest against both sides of the exposed joint (lateral and medial epicondyle).

– For EPIDYN AKTIV®, attach the band in a circular fashion under the elbow. First pull the band from the inside over the upper arm.

– Adjust the band pressure depending on your needs.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

We recommend that EPIDYN® and EPIDYN AKTIV® be washed gently in lukewarm water with detergent for delicate items and that it then be air dried (do not dry on a heater).

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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Wszystkie wyroby SPORLASTIC GmbH podlegają kontroli w ramach naszego systemu zarządzania jakością.