MALLEO-HiT® FS SPORLASTIC

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INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

MALLEO-HiT® FS is an ankle bandage with silicone friction pads, and it is used to relieve stress and compression on the ankle joint and to stabilise it. Thanks to targeted friction, the silicone pads with their integrated dimple pattern create an improvement in the blood circulation in the joint area. MALLEO-HiT® FS helps the foot to perform its wide range of tasks by supporting the dynamic stress operations when walking and standing, and by stabilising the structure of the foot.

The special guide strap takes up the stress in a controlled way and converts it into dynamic movement. This means that not only is standing more secure and walking more elastic, but the foot and body become less tired.

INDICATIONS

- Instability in the foot
- Stress complaints in the leg
- Instability in the ligaments of the foot and ankle joint
- With chronically unstable ankle joints
- Sport stresses (condition after twisting trauma)
- Osteoarthritis of the upper and lower ankle joints

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING

- The MALLEO-HiT® FS can be worn on the right or left foot.

- Pull the bandage over the foot, until the heel sits anatomically correctly in the knitted heel part. The silicone profile inserts should then be positioned behind and below the internal or external malleolus (Fig. 1). Then pull the outer strap under light tension over the outside of the foot onto the back of the foot and hold it there under light tension (Fig. 2).

- Then pull the inner strap under light tension around the inner arch of the foot and fasten it onto the Velcro of the outer strap. (Fig. 3) The tension should be pleasant and not too strong; the straps should run at an upward angle from the sole of the foot right up to the internal or external malleolus (Fig. 4).

- Finally walk a few steps and correct the tension of the straps until comfortable. Should you not be putting stress on the foot for some time (for example when sitting), reduce the tension of the straps or open them. You should always however select the strap tension, which ensures that the necessary stabilisation of the foot is achieved. Regularly perform foot massage and foot gymnastics.

COMFORTABLE TO WEAR

MALLEO-HiT® FS's special stitching in the instep area makes for an almost crease-free fit during heel-toe walking, substantially improving comfort when wearing.

IMPORTANT

The product is to be used by one patient only.
Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.
For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

GENERAL NOTE

The use of products involving direct exposure of the skin to an elastomer (e.g. a silicone pad) might possibly lead to blisters on the skin if the product is applied too tightly or if it is shifted on the skin at high friction. In this case, we recommend to discontinue the use of the product. Sweat may be generated under the silicone and, in rare cases, this can lead to the body reacting to its own sweat and cause corresponding allergic skin changes. Sporlastic is not aware of any allergic reactions related to the materials. If you experience any problems, please contact your doctor or the supplier of the medical aid.

CARE

Wash MALLEO-HiT® FS in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. The enclosed heel pad can be cleaned with a moist cloth.

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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