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INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

FUNCTION

The combination of the two pads relieves pressure topically on the Tuberositas tibiae, and the moulded U-shaped silicone section with the opening underneath relaxes the patellar tendon **1**.

INDICATIONS

- Schlatler's disease
- Traction apophysitis
- Prominence of the Tuberositas tibiae caused by abnormal ossification
- Initial tendinosis on the Tuberositas tibiae

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped shaped body fitting.

FITTING (right seat see picture)

- Pull the bandage up over the knee so that the upper arc of the semicircle is just above the kneecap.
- The lower cushion then envelops the Tuberositas tibiae.
- Now fasten the correction straps by tightening the Velcro fasteners accordingly.

INNOVATION MAKES IT COMFORTABLE TO WEAR

Morbus-Schlatter is very comfortable to wear because of the almost crease-free ideal fit achieved by the concertina effect in the popliteal cavity. The essence of the innovation is an oval intarsia element. When the knee is stretched, the intarsia element extends; when the knee flexes, the stitching contracts - without any gathering of material in the popliteal cavity. This has noticeable medical benefits for the patient.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in conjunction with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

Wash Morbus-Schlatter in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. The enclosed heel pad can be cleaned with a moist cloth.

TIP: Slackening tension of the knitted fabric while it is being worn is revived by washing. The bandage should therefore be washed frequently.

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