



Adjustable Double Adaptor

N-D231

Instruction for Use





Description

A Double Adaptor with an Adjustable range between 60mm to 75mm.

ASSEMBLY AND FITTING


1. To increase Adaptor overall length undo Screw A, then rotate bottom receiver pyramid end anti-clockwise to increase length up to 15mm, then to secure in position re-tighten screw A to torque shown.
2. Alignment can be retained by adjusting 2 adjacent screws.
3. The rotational direction can be adjusted as well (it changes length 0.5mm by 90° rotation).
4. DO NOT modify the products such as changing the maximum length (75mm).

TECHNICAL DATA

Product Weight	Patient Weight Limit	Activity Level
170g	100kg	High



To achieve increased angulation, 2 longer screws are supplied.

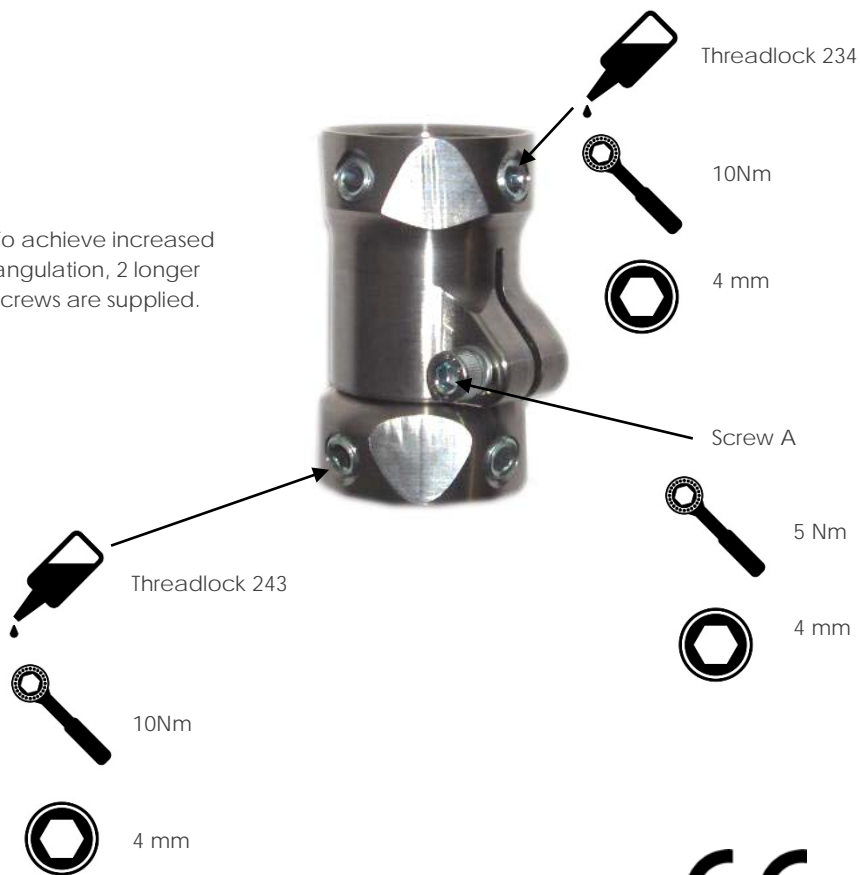



WARRANTY

Components are covered by a 12 months limited warranty. The warranty requires fitting and use as set out in these instructions. The warranty is void if fitting and routine care instructions are not followed.

