Art. 07294

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Art. 07294

OMO-HiT® ABDUCTION

Shoulder joint orthosis for abduction positioning

INSTRUCTIONS FOR USE (GB)

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any questions, please consult the doctor treating you or your nearest stockist, or contact us directly.

Indications

- Subacromial bursitis/impingement
- Mobilisation after shoulder stiffness
- Pre- and post-operative following trauma and/or joint replacement
- In conservative treatment of shoulder-arm injuries, in particular also for injuries/ruptures of the rotator cuff
- All conditions which require fixing of the arm in the abduction position

Contra-Indications

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.

Utilisation requires a prior check-up by your doctor, therapist or technician for:

- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

Side effects

To date, there have been no reports of serious general side effects with proper use and fitting. Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restrictive, consistently shaped body fitting.

Cleaning instructions

Hand wash with mild soap in lukewarm water, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation and deteriorate material. The Abduction pillow can be wiped with a damp cloth – do not immerse Abduction Pillow in liquid.

APPLICATION INSTRUCTIONS

1. Detach quick release buckle of shoulder strap. While supporting the injured arm, slide the forearm into the sling. Wrap the adjustable shoulder strap around the neck and shoulders and attach the quick release buckle.

2. The shoulder strap has two adjustable ends. Adjust the shoulder strap to the proper length.

3A. The shoulder pad may be adjusted by removing it from the shoulder strap and repositioning it over the neck and shoulder.

3B. Attach the closure strap across the top of the sling near the elbow so that it holds the arm securely in the sling.

3C. Attach the thumb strap to the open end of the sling between the thumb and fingers.

4. Attach the abduction pillow to the inside of the brace.

5. Wrap the pillow strap around the waist and attach the quick release buckle. Tighten waist strap and secure.

6. Attach the exercise ball to the end of the pillow.

7A. Straps may be shortened if too long. Remove the tab from the end of the shoulder and/or waist strap.

7B. Trim to desired length using scissors. Apply as described above.