
OMO-HIT®
SPORLASTIC

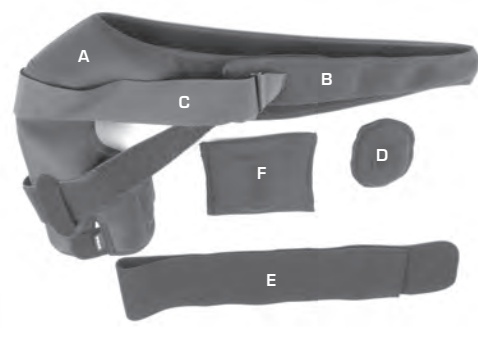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INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest stockist or contact us directly.

FUNCTION

OMO-HIT® is a shoulder bandage for conservative or post-operative treatment of the shoulder. Close yet very specific belt lacing achieves the dynamic, immobilising effect of this bandage which is supplied with a loop-pile, silicone butterfly pad and a cold/hot pad. The bandage is supplied with a shoulder sling for short-term immobilisation.

INDICATIONS

- After-treatment following shoulder dislocation or semi-luxation
- Distortion, contusion
- Immobilisation in the event of various upper-arm fractures
- Shoulder blade fracture
- After-treatment of rotator cuff injuries
- Bursitis subacromialis
- Post-operative treatment following arthroscopic interventions

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING

Applying the OMO-HIT® shoulder/upper arm fixation:

– **FIG. 1:** Apply the OMO-HIT® in an open state, as indicated here.

– **FIG. 2:** The OMO-HIT® bandage is positioned in such a manner that the top seam of the shoulder section (A) is located on the middle of the shoulder.

– **FIG. 3:** As indicated here, the Velcro fastener silicone butterfly pad (D) is individually positioned as instructed by the doctor or orthopaedic technician.

– **FIG. 4:** Finally, close the OMO-HIT® bandage with the Velcro fastener (B) in such a manner that both parts are joined together above each other.

– **FIG. 5:** After the previous steps have been carefully completed, the immobilising belt (C) is threaded through the loop on the Velcro fastener (B) and secured to the shoulder section (A). It may be necessary to shorten the immobilising belt (C) with scissors to adapt it to the given body size.

VARIATIONS:

– **FIG. 6:** With some indications the doctor may prescribe short-term immobilisation of the shoulder and/or elbow joint. For this purpose the supplied shoulder loop (E) is attached to the Velcro fastener on the back of the shoulder section (A) to secure it underneath the immobilising belt (C) and adjusted in such a manner that the arm – if not prescribed otherwise by the doctor – is fixed in a 90 degree position.

– **FIG. 7:** The cold/hot pad (F) is positioned as prescribed by the doctor or orthopaedic technician and secured in place with the Velcro fastener. Cold/hot treatment is carried out according to the doctor's instructions.

Warning: Please observe the instructions on the cold/hot pad (F); (the textile cover has to be removed).

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper

use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

We recommend that OMO-HIT® be washed gently in lukewarm water with detergent for delicate items and that it then be air dried (do not dry on a heater).

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Wszystkie wyroby SPORLASTIC GmbH podlegają kontroli w ramach naszego systemu zarządzania jakością.
Alle Produkte von der SPORLASTIC GmbH sind unterworfen dem Qualitätsmanagement-System.